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These instructions ensure a maximum service life of your FINO VITESSE micromotor.

Never operate, transport or store the FINO VITESSE hand piece without mounted tool! Otherwise the tongs and the quick gripping mechanism can be damaged. Always use the test drill supplied when replacing or cleaning the tongs! Clean the hand piece every week. For this purpose remove the front cap and carefully clean the parts with a brush. Do not use compressed air! Remove the tongs as described in the instructions (with inserted 2,35 mm drill!) and clean them in the ultrasound unit. Then remount the cleaned and dried tongs in the hand piece as described in the instructions and attach the tongs hand-screwed! Following this, put the cap back on hand-screwed.

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These instructions ensure a maximum service life of your FINO VITESSE micromotor.

Never operate, transport or store the FINO VITESSE hand piece without mounted tool! Otherwise the tongs and the quick gripping mechanism can be damaged. Always use the test drill supplied when replacing or cleaning the tongs! Clean the hand piece every week. For this purpose remove the front cap and carefully clean the parts with a brush. Do not use compressed air! Remove the tongs as described in the instructions (with inserted 2.35 mm drill!) and clean them in the ultrasound unit. Then remount the cleaned and dried tongs in the hand piece as described in the instructions and attach the tongs hand-screwed! Following this, put the cap back on hand-screwed.

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