FINOGLOSS

Glanzlösung 02288/ 02289

Deutsch

Gebrauchsinformation

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User Information

1. Fields of application

Crystal-clear polishing aid for resin prosthetics. All resin works get a rich lustre. The surface of the denture obtains a microscopic fine protection and an intensive sealing.

2. Safety precautions

Danger

H225 Highly flammable liquid and vapour.

H319 Causes serious eye irritation.

H336 May cause drowsiness or dizziness.

2.1 Prevention:

P210 Keep away from heat/sparks/open flames/hot surfaces. No smoking.

P280 Eye protection/Wear protective gloves.

2.2 Response:

P303+P361+P353

IF ON SKIN (or hair):

Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313

If eye irritation persists: Get medical advice/attention.

2.3 Disposal
P501 Dispose of contents / container in accordance with local / regional / national / international regulations.
Waste codes/waste designations according to EWC/AVV 07 06 99

3. Processing instructions

Apply a few drops of the polishing aid onto the denture to be polished with a cotton cloth or a polishing disc. Let the high-gloss solution evaporate and subsequently polish the workpiece as usual.

4. Storage

P403+P233
Store in a well-ventilated place.
Keep container tightly closed.

5. Physical data

Optical characteristic: clear solution
Density: 0.92 g/ml
Contains: Ethyl acetate

6. Delivery forms

FINOGLOSS high-gloss solution
250 ml 02288
100 ml 02289

7. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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