FINO QUICK PLUS
Modellsystem 10730

Deutsch

Gebrauchsinformation

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User Information

1. Fields of application

High precision, reusable model system that makes drilling of pinholes, placing of pins and fabrication of splitcasts redundant. FINO QUICK PLUS model system ensures the optimum processing of saw models in the crown and bridge technique.

2. Safety precautions

10735/10739 Separation resin to plaster:

Danger

H225 Highly flammable liquid and vapour.

H304 May be fatal if swallowed and enters airways.

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H336 May cause drowsiness or dizziness.

H410 Very toxic to aquatic life with long lasting effects.

2.1 Prevention:

P210 Keep away from heat/sparks/open flames/hot surfaces. No smoking.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

2.2 Response:

P301+P310

IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician.

P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.

Continue rinsing.


P331 Do NOT induce vomiting.

Contains heptane and isomers of 2-propano.

3. Processing instructions

3.1 Denomination of the different components (fig. A)

1. FINO QUICK PLUS, pin base plate large and small
2. FINO QUICK PLUS, splitcast plate (included in each pin base plate large and small)
3. FINO QUICK PLUS, sleeves large and small
4. FINO QUICK PLUS, ejection plate (included in pin base plate large and small)
5. FINO QUICK PLUS, detaching tool

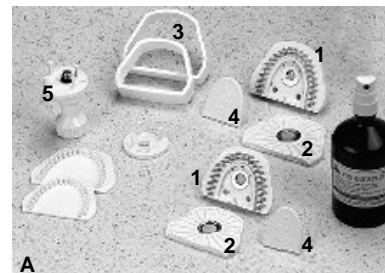3.2 Preparation of impression

First of all the correct size of the pin base plate is selected according to the impression. Take care of correct fitting. In order to obtain a flat die arch and to facilitate the later removal, the impression is trimmed in the vestibular area and in the palatal area in a way that deep undercuts are bevelled. The surface treatment of the impression is done according to the directions of the manufacturer.

Mark middle of model and other possible reference lines for later positioning.

3.3 Placement of putty

Substitute missing areas or air bubbles in the impression with putty.

3.4 Preparation of the pin base plate

The fitting sleeve is placed on the pin base plate. The pin base plate is only insulated with the included FINO QUICK PLUS /EVO Separating Agent by spraying on a thin even film.

Important: After drying remove the insulation carefully from the outer margin.

3.5 Casting of the model

The impression is poured as usual. Fill the pin base plate with plaster just above the pin level.

Place impression on the pin base plate and position according to the reference markings.
Adjust occlusal surface parallel to the pin base plate if possible. Remove excessive plaster.

3.5 Lifting of the tooth arc

After setting of the plaster the impression is divested and the white sleeve is removed.

The white sleeve is removed and the pin base plate is taken off the splitcast plate to get access to the holes for the detaching device.


The detaching device is placed into the holes on the bottom of the pin base plate designed for this purpose.

By screwing at the detaching device the die arch is carefully taken off, whereby the lid of the model system comes loose. Please pay attention that canting of the die arch must be avoided.

3.7 Grinding of the die arch

The vestibular aspect of the die arch is ground with a dry or wet model grinder in a way that the first row of pinholes is uncovered. If a wet model grinder is used any plaster sludge should immediately be removed from the pinholes before the model is again placed onto the pin base plate.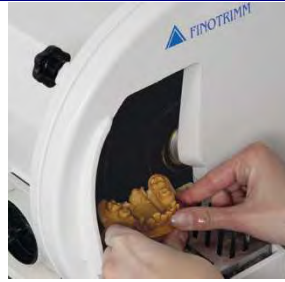

For the preparation of the oral aspect a die arch cutter or a pointed cutter are best suitable. In any case it is important that the die arch is only reduced up to the boundary line, which is marked on the ejection plate as little groove. It is designed to take off the separate segments.


Finish the die arch as required.

3.8 Finishing of the model

Saw die arch. You do not care about the position of the pin holes. The segments always have enough retention on the plate.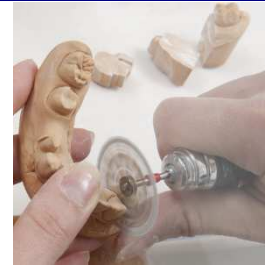

Remove plaster dust thoroughly with compressed air and a brush. The model is now finished. The marking of the preparation margin can now begin. Before the non-sawn die arch is repositioned, the pin base plate must be cleaned thoroughly with warm water and common rinsing agents. A soft hand brush with plastic bristles is best suitable for this purpose.

Because the split-cast base already belongs to the pin base plate subsequently the articulation can start.

3.9 Cleaning of pin base plates

Please note:

Please clean the pin base plates only with FINO QUICK PLUS /EVO Cleaning Spray (item-no. 10749) or the FINO GIPS-EX QP Plaster Remover (item-no. 10285, 10286). Do not clean in an ultrasonic bath or with other caustic cleaning agent. By using other cleaning agents or separating agents the pin base plates as well as split-cast plates may get cracked or crumbly.

4. Storage

10735/10739 Separation resin to plaster:
Store at room temperature of 25 °C.

5. Physical data

No further data.

6. Delivery forms

FINO QUICK PLUS Model System

Contents of introduction kit:

Pin base plate	large	2x	10731
Pin base plate	small	3x	10732
Sleeve	large	1x	10733
Sleeve	small	2x	10734
Detaching tool		1x	10737
Separating agent	100 ml	1x	10739
Archiving plate	large	2x	10740
Archiving plate	small	3x	10741

6.1 Accessories

Pin base plate	large	10731
Pin base plate	small	10732
Sleeve	large	10733
Sleeve	small	10734
Separating agent	500 ml	10735
Detaching tool complete	large	10737
Detaching tool complete	small	10738
Separating agent	100 ml	10739
Archiving plate	large	10740
Archiving plate	small	10741
Archiving plate de luxe,	large	10742
Archiving plate de luxe,	small	10743
Cleaning pin		10748
Cleaning spray	400 ml	10749

7. Garantie

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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