**PALA**  
**Aislar**<sup>®</sup>

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**GB**   User Instructions

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**E**    Modo de empleo

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**S**    Bruksanvisning

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## **GB**   User Instruction

Insulation, free of formaldehyde

### **Area of application in Dental Laboratories**

The insulation for plaster models for the resin technique. Aislar forms a very thin smooth and even coating which guarantees highest precision of denture fit. The insulation coating protects against non-desired influx at the resin and results in a smooth surface. Finishing time is reduced to a minimum.

The insulation can be applied onto warm and cold plaster. A second application of the insulation gets better results.

### **Advice**

Not for use in the mouth.

### **Handling**

The wax must be thoroughly scalded. Place the required quantity into a jar and apply with a brush. Coat the plastertwice with insulation after scalding. No residue should remain in grooves and furrows. Acrylic teeth and clamps should not be wetted. The insulation coating must be absolutely dry prior to pouring the resin dough. With dried out models all plaster parts must be sufficiently watered prior to insulation. The watering time is subject to the degree of dryness and is approximately 5–15 min. Watering prevents evaporation of air from plaster into the resin during polymerization under pressure. Surplus Aislar already poured out must not be returned to the original bottle.

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### **Storage advice**

Do not use material after expiry date. Do not store above 25°C. Avoid direct exposure to sun. Close container immediately after use. Keep out of reach of children.

### **Supply units**

Single pack  
80 ml  
500 ml  
2 x 1000 ml

® = registered trademark of Heraeus Kulzer GmbH

### **Caution:**

U.S. Federal law restricts this device to sale by or on the order of a dentist.

Distributed in USA/  
Canada exclusively by:  
Heraeus Kulzer Inc.  
99 Business Park Drive 10504  
Armonk, NY USA

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