 **FINOPASTE**
Silikonknetmasse
15180/15185/15186

Härterpaste
15184

Deutsch

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English

Instructions for use

**Please read carefully before use
Keep for referral purposes**

1. Fields of application
On C-silicone base. For fabrication of repair models, block-outs, linings and matrices. Only for extraoral use!

2. Safety precautions

 Warning
H315 Causes skin irritation.
H319 Causes serious eye irritation.
H373 May cause damage to organs through prolonged or repeated exposure.
H413 May cause long lasting harmful effects to aquatic life.

2.1 Prevention
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P273 Avoid release to the environment.

2.2 Response
P302+P352 IF ON SKIN: Wash with plenty of soap and water.
P332+P313 If skin irritation occurs: Get medical advice/attention.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P337+P313 If eye irritation persists: Get medical advice/attention.
P314 Get medical advice/attention if you feel unwell.

3. Processing instructions

3.1 Dosing
Dose a circular measuring spoon of silicone modelling material and wipe off excess. Spread dosed amount on hand and dose a strand of curing paste per circular diameter. Mixing ratio silicone modelling material : curing paste 11.0 g : 0.3 g.

3.2 Mixing
Fold the silicone modelling material and knead both components well for 30-45 s until a homogenous mixture (colour) has been achieved. Process the silicone modelling material.

3.3 Curing
Curing time at least 5-7 min at room temperature. Curing can be accelerated or slowed down through over or underdosing of the curing agent.
The curing time is accelerated by longer kneading.

4. Storage
P404 Store in a closed container.
P411 Store at temperatures not exceeding 23 °C/50 °F.
P410 Protect from sunlight.

5. Physical data
Mixing time 30-45 s; processing time 90 s; curing time at room temperature 5-7 min; hardness 66 Shore A; colour silicone modelling material (base): light grey, colour curing paste (catalyst): blue.

6. Delivery forms

FINOPASTE		
Silicone putty	1400 g	15180
	4500 g	15185
	15000 g	15186
Hardener	40 ml	15184

7. Guarantee
Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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