Deutsch

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**Operating Instructions**

**1. Fields of application**

For thorough and gentle cleaning of many different objects in dental laboratories. For removal of polishing residue and other contamination via ultrasonic oscillations.

**2. Safety precautions**

Caution: Failure to comply with this hazard information can lead to serious injury of the operator(s) and/or other persons.

The unit must only be used for the intended purpose described in the scope/field of application.

Do not operate the unit with wet or damp hands. There is a risk of electric shock, which could potentially be fatal.

Never put your hands in the oscillating fluid. Never operate the unit without fluid in the oscillating tank. Otherwise there is a risk that serious damage could be caused. Always fill the oscillating tank with fluid up to the recommended minimum fill level. During the cleaning process the fluid is heated up and evaporation increases. As a result, the unit must always be operated with a cover to prevent excessive evaporation. Only use cleaning agents recommended by the manufacturer, e.g. FINOSONIC cleaning concentrate.

Never fill flammable liquids like petrol or other solvents into the unit. There is a risk that these could ignite.

Do not use chemicals that contain or separate into chloride ions, such as domestic cleaners or disinfectants.

Only empty the unit when switched off.

Only transport the unit when empty.

Do not use the unit if any damage is evident to the housing, cables or any other parts of it.

**Note:**  
Always use a cleaning basket.  
Never place any objects directly onto the floor of the tank, as this will cause the stainless steel tank to wear more quickly.

**3. Processing instructions**

**3.1 Description**

FINOSONIC PRO has a timer from 1 to 30 minutes. The integrated surface heating of 100 W can be additionally switched on if required. It will automatically switch back off again when the bath reaches a temperature of 60°C. Switching on the heating achieves better and faster cleaning results.

**3.2 Getting started**

Before first operation check the unit for transport damages. Claim any transport damages immediately with the supplier.

Place the unit on a dry and level surface. Make sure that no moisture can enter the unit from underneath. Do not obstruct the flow of air underneath the unit.

**3.3 Handling**

Thoroughly rinse out the oscillating tank before using it for the first time. Connect the ultrasonic unit to the mains power supply. Only fill the oscillating tank with ultrasonic cleaning fluid that has been prepared in accordance with the manufacturer's instructions. Place the objects that are to be cleaned in the basket insert. Hook the basket into the perforated lid insert and immerse it in the oscillating tank. The cleaning fluid level must be 1 to 2 cm above the objects.

**3.3.1 Cleaning duration**

Set the cleaning duration as short as possible. Set the cleaning duration from 1 to 30 minutes depending on how dirty the object is and depending on the cleaning fluid used. A default cleaning time of 5 minutes is preset. The duration can be adjusted via the "Time" button and the arrow key to the right of it. Once the cleaning time has been set, start the cleaning process with the "On/Off" button.

**3.3.2 Heating**

If required, switch on the heater via the "Heater" button.

Heating the fluid improves the cleaning effect. The best cleaning results are obtained with a bath temperature of 50 to 60 °C.

In order to protect the electronic components, the heater automatically switches off when the bath temperature reaches 60 °C (overheating protection). A red lamp lights up, and the unit cannot be switched on. After around 20 minutes the lamp lights up green and the unit is ready for operation again.

**3.3.3 Degas function**

Newly prepared cleaning fluids must be degassed before they are used in order to accelerate the cleaning process. To do this, first select the cleaning time, then press the "Function" button. Press the "On/Off" button to start the degassing process. After 90 seconds the unit will switch over to the standard cleaning process.

**3.4 Service and maintenance**

Maintenance-free.

Any repairs to the unit must only be performed by an authorised and suitably qualified and experienced technician. The user is liable for any damages resulting from unauthorised repairs.

Always disconnect the unit from the mains supply before cleaning it or performing any maintenance on it. Do not spray the unit with water, immerse it in water or expose it to splash water.

**Oscillating tank:**

The oscillating tank is a wearing part as it is continuously exposed to cavitation during operation. Due to the movement of the fluid, remaining dirt particles can rub against the surface and cause damage to it. For this reason, regularly give the oscillating tank a thorough rinse with water and rub it dry with a soft cloth. Remove edges and residue with standard commercially available stainless steel cleaning agents without scouring additives. Do not use steel wool or scrapers for cleaning.

Metallic objects left in the unit can cause it to rust. If this happens, remove any rust immediately with a soft cloth and a stainless steel care product to prevent pitting corrosion.

**Housing:**

Only use standard commercially available cleaning agents without scouring additives. Only wipe down the housing from the outside with a damp cloth, and then leave it to dry or rub it dry.

**4. Storage**

Store the unit with the lid on to protect it against moisture.

**5. Technical data**

external 24.5 x 31.5 x 20.5 cm; internal 150 x 245 x 75 mm; 220-240 V, 50/60 Hz; 170 W; ultrasonic frequency 35 kHz; cleaning time variably adjustable 1-30 min; tank volume 2.5 l; filling volume 2.1 l; 2.9 kg.

**6. Delivery forms**

FINOSONIC PRO ultrasonic cleaning unit with cleaning basket

1 pc. 51868

**7. Guarantee**

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

**8. Disposal in accordance with EU guideline 2002/96/EU**

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

**8.1 Registration Number**

WEEE-Reg.-No. DE 75542610

**Important:**

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document.

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