REF 80016

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REF 80016

ENG Instructions for use

Please read carefully before use
Keep for referral purposes

R_x only Only to be used by qualified
personnel!

1. Fields of application

Headband magnifier with various magnification options and individual headband adjustment.

2. Safety precautions

Risk of fire if handled or stored inappropriately due to burning glass effect of the lenses.

Never subject optical lenses to direct sunlight without a cover.

Risk of dazzling and injury!
Do not look into the sun with optical devices or other bright light sources.

3. Processing instructions

3.1 Preparation of light source

Pull LED module to the stop and release by pressing the locking mechanism (see Fig. 1). Remove LED module. Open closing cover. Insert 3 batteries type AAA (see Fig. 2). The LED module can also be used separately (see Fig. 3).

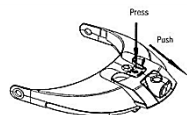
Fig. 5

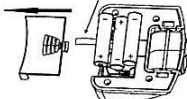
Fig. 6

Fig. 7

3.2 Inserting/removing the magnifying glasses

Insertion: align magnifying glasses with their convex side facing away from the eye and insert in holder. Serial connection is possible. (see Fig. 4).

Removal: pull magnifying glasses **parallel** from the holder to avoid damaging them.

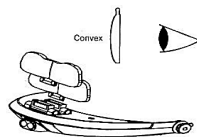
Fig. 8

3.3 Magnification options

Magnification	Working distance
1.0x	approx. 300-350 mm
1.5x	approx. 240-260 mm
2.0x	approx. 215-235 mm
2.5x	approx. 205-225 mm
3.0x	approx. 160-180 mm
3.5x	approx. 135-145 mm
4.0x	approx. 125-135 mm
4.5x	approx. 110-130 mm
5.0x	approx. 100-120 mm
5.5x	approx. 80-95 mm
6.0x	approx. 60-85 mm

3.4 Service

Clean with a soft cotton wool or linen cloth. Only wipe gently over the lens surfaces. Rinse with lukewarm water in case of stronger contamination. Do not use alcohol-containing or organic solvents.

4. Storage

P410 Protect from sunlight.
P412 Do not expose to temperatures exceeding 50 °C/122 °F.

5. Delivery forms

FINOZOOM headband magnifier complete with removeable LED module; 5 magnifying glasses (1.0x, 1.5x, 2.0x, 2.5x, 3.5x) and 3 batteries type AAA.

6. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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