Paladur®

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso

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66067352/01

## **GB** Instructions for use Paladur®

**Medical Device – for use by dental health care professionals only.**

**Intended purpose**

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the associated risks must be weighed carefully against the benefits taking into consideration the unborn child or infant. KULZER medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

**Product description**  
Paladur is an autopolymerising prosthetic resin for all the indications listed below. It consists of a powder component in a range of colours (pink, pink veined, R50 veined, clear) and liquid. Prosthetic resin for casting fixed and removable prosthetic devices that cures in thin layers on contact with air without technical effects.

**Composition**  
Cold-curing polymer, consisting of powder and liquid. Main powder component: Methyl methacrylate copolymer. Main liquid components: methyl methacrylate, 2-(2H-Benzotriazol-2-yl)-4-methylphenol. The catalyst system contains tertiary amine. Paladur is cadmium-free.

**Indications**  
For fixed and removable prosthetic devices. Can be used for:

- Restoring function
- Extension
- Repairs
- Indirect partial and total relines

**Contraindications**  
Direct lining with Paladur is contraindicated. The use of this product is contraindicated in case of known or suspected allergies against components of this product. Do not use in case of known or suspected allergy against (meth) acrylate compounds or benzoyl peroxide.

**Side effects**  
This product or one of its components may in particular cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer. Hypersensitive reactions (allergies), local sensory disturbances, taste disturbances and irritation of the oral mucosa have been described in isolated cases. After processing, all MMA/PMMA-based denture materials always contain a permitted, small proportion of residual monomer (<4.5%). For further risk reduction of intolerance reactions, the denture has to be stored in tepid water for at least 12 hours before placement.

**Handling**

Dental stone is recommended for the production of models.

When performing extensions or repairs, use hot water to remove residual wax carefully from the resin surfaces to be bound. Roughen a wide area on the resin parts to be bound (e.g. when repairing a break) to ensure mechanical retention is ensured. The wet plaster model is insulated twice using thin layers of Aislar® (see Aislar® instructions for Use). This is the only way to ensure perfect surface quality between Paladur and the model. To improve the bond between Paladur and prosthetic resins or the pro-fabricated teeth, the contact surfaces are roughened with a coarse diamond bur (remove grinding dust) and wetted with Paladur®. Apply Paladur® twice using a brush (without metal ferrule) and allow to set for 30 seconds each time. After the second application the adhesive remains active for 10 minutes.

**Amounts required**  
The recommended mixing ratio is 10 g powder : 6 ml liquid. Introduce the liquid to the mixing beaker, add the appropriate quantity of powder rapidly and stir for up to 15 seconds to make homogeneous paste. To remove air bubbles, swirl the mixing beaker. Working time depends on ambient temperature. The drippy liquid paste can be worked immediately. Apply Paladur evenly and once the plastic consistency has been attained, do not mix or model further.

**Polymerisation**  
The curing time at room temperature 23°C (73°F) is approx. 9–12 minutes. The optimal polymerisation without formation of bubbles is achieved with the Palamat® elite / Palamat® practc for 10 minutes, at a water temperature of 55°C (131°F) and a pressure of 2 bar. When working without a pressure polymerisation unit, apply larger amounts of Paladur in layers. Each layer must be allowed to cure and cool completely before new paste is applied.

**Trimming and polishing**  
After closing and re-occlusion the prosthesis is carefully removed from the model and trimmed with cross-toothed tools. Before polishing with pumice, the prosthesis is smoothed with sandpaper of decreasing grit size.

**Repair**  
All cold-curing Pala prosthetic resins from Kulzer are compatible with one another and can be used for restoration/addition and repairs. The heat-curing Pala prosthetic resins from Kulzer can also be restored, added to and repaired with all cold-curing Pala prosthetic resins.

**Storage advice**  
Do not store above 25°C (77°F). Do not use after expiry date. Avoid direct exposure to sun. Reseal carefully after removing product.

**Warnings and safety instructions**  
May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash plenty of water and soap. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and

easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Highly flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources – no smoking. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Toxic to aquatic life with long lasting effects. Flash point: 10°C.

**Application notes**  
Examine the integrity of the primary packaging before first use. Damaged products must not be used. When positioning teeth in wax, do not use alcohol based separating agents (separation of gypsum and wax), as this may cause white discoloration on the basal surface of the prosthesis.

Please quote batch number and article number in all correspondence about the product. According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

**Disposal information**  
Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Safety data sheets and more information are available at our website [www.kulzer.com](http://www.kulzer.com)

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® = registered trademark of Kulzer GmbH
® = registrovaná ochranná známka spoločnosti Kulzer GmbH