REF 10380

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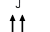
REF 10380

ENG Instructions for use

Please read the entire document before using the Flocculator.

1. Intended purpose

Binds plaster, quartz and phosphate-bonded investment materials.

2. Indication

Flocculant prevent the closure of sewage pipes, reduce the smell of rotting and facilitate the cleaning of the separator.

2.1 Contraindication

Not known.

In case of allergic reactions to product components, sensitive individuals cannot be excluded.

3. Fields of application

3.1 General

Only for processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group

The target user groups for Flocculator are:

- Dentists
- Dental technicians

The patient target group for Flocculator includes patients receiving as part of dental treatment.

3.3 Cleaning

Flocculator are products intended for single use. They do not require cleaning.

Flocculator are not intended for disinfection.

4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

Attention

H319 Causes serious eye irritation.

4.1 Prevention

P264 Wash hands thoroughly after handling.

P280 Wear protective clothing/eye protection/face protection.

4.2 Response

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

5. Processing instructions

Place a tab in the clip-on basket and hang it in the plaster separator.

The tab must lie below the water surface and must not be directly exposed to the plaster-contaminated water inflow.

For multi-chamber separators, hang the tab on the overflow to the second chamber.

Replace the tab weekly and every time the plaster separator is emptied.

6. Storage

P232 Protect from moisture.

P233 Keep container tightly closed.

P402 Store in a dry place.

 Protect against frost.

Do not store together with potent acids.

7. Physical data

No further data.

8. Delivery forms

Packaging units for Flocculator are:

- floculant
- Instructions for use

The scope of delivery of each individual product consists of:

FINOPUR TABS floculant, 21 tabs with clip-on baskets

1 pack. 10380

floculant are shown in the illustration:

Flocculator are supplied without accessories.

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent

of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product
070104 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the manufacture, formulation, supply and use (MFSU) of basic organic chemicals; other organic solvents, washing liquids and mother liquors; hazardous waste.

12. Symbols explanation

12.1 Manufacturing

 Item numbers

 Lot code

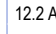
 Shelf life

 non-sterile

 Unit in the packaging

 Manufacturer

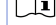
12.2 Application

 General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

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