FINO MIXERY
Vakuum-Anmischgerät
10562

Deutsch

Gebrauchsinformation

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User Information

1. Fields of application

Easy to operate and rugged unit for the bubble-free mixing of plasters, investment materials, silicones or other impression materials.

2. Advantages

- No compressed air connection required
- Selectable speed
- Can be programmed
- Maintenance-free
- Powerful mixer

3. Safety precautions

Repairs of this product should only be done by DT&SHOP an authorised workshop. The user has to stand good for damages caused by unauthorized/ arbitrary repairs. The unrestricted conditions of delivery and guarantee of DT&SHOP apply for this unit.

- Do not touch with wet hands.
- The unit is equipped with a precise power circuit. Therefore it is important to avoid impacts and knocks because it may lead to mechanical problems.
- Do not use damaged or twisted cable or plugs because this may lead to lose contacts.
- Apart from this user information the national directives pertaining accident avoidance must be observed.
- Before any service, cleaning or repair switch unit off and pull mains plug.
- Do not connect the mixing bowl while the mixer is moving.
- Operation should only be done by qualified personnel.

4. Processing instructions

4.1 Installation information

FINO MIXERY comes for wall-mounting as a standard. To use it as a tabletop unit additionally the tabletop stand item-no. 10560 is required.

4.2 Installation directive

- For mounting choose a straight, carrying wall.
- Make two horizontal marks exactly 80 mm apart.
- Drill holes at the marked points and put in the included plugs.
- Screw in the included screws. The screw heads must protrude 4 mm.
- Make sure that the screws are tightly anchored and hang up the unit.
- Mark the position of the fixation screw below the unit.
- Remove the mixer.
- Again drill a hole and put a plug in.
- Hang up the unit again.
- Attach the lower screw with a screw driver.

After mounting the unit is ready-to-use. Check whether the voltage indicated on the type label of the unit is the same then the available mains voltage. Connect the unit plug first to the vacuum mixing unit and subsequently in the earthed socket. Place the cable in a way that it is no tripping hazard.

4.3 Getting started

Before starting the unit check whether the power cable and the unit are operationally safe. The power cable must not show damages like cracks, jams or a damaged insulation. Should you detect defects immediately take the unit out of operation and make sure that it cannot be put into operation again. Let it be maintained by an authorised service centre.

4.4 Operation

4.4.1 Selection of display

Switch unit on using the main switch. The upper display shows the rotational speed. The lower display shows the operation time in seconds. You can change the displayed values with the arrow keys ▲/▼. To change between the displays push the □ key.

4.4.2 Mixing

Put the desired material in the bowl and mix it manually with a spatula. To start the mixing process close the mixing bowl with the lid. Place the bowl in the coupling on the lower side of the unit and turn it slightly until it catches in the coupling.

Press the Start/Stop key ⏻, to start the vacuum pump. Lightly pushing in the middle of the bottom of the bowl towards the unit is the fastest way to generate the vacuum. After the needle of the vacuum gauge is in the yellow area there is enough vacuum to keep the bowl in the coupling. During the mixing process the needle must be in the green area. The process can be stopped at any time with the Start/Stop key ⏻.

4.4.3 Removal of bowl

- To remove the bowl you hold it and press the black key (●).
- Do not remove the mixing bowl while the mixer is moving.

4.4.4 Programming of the unit

The FINO MIXERY offers the option to store 4 individual programs (Selection of mixing speed and time). Choose the desired values as described in section 4.4.1 by using the respective keys ▲/▼, □.

For storing you press the key **P** on the input field. For storing the 4 different programs you must press the key **P** so many times until the selected program number is reached. (P1= press 1x **P**, P2=press 2x **P** etc.).

To enter the program number press the key **P** for 3 seconds.

4.4.5 Select program

To select the stored programs press key **P** in the input field. The respective entries are shown in the displays.

4.4.6 Modify program

Repeat process as described in section 4.4.4. Thereby the previous entries are overwritten.

4.5 Service and Maintenance

Before maintenance switch the unit off and unplug from the socket. For the cleaning of the unit only use dry or slightly wet cloth.

The unit must not be sprayed or submersed in water. Do not use solvents for the cleaning of the unit. The mixing bowls can be cleaned in an ultrasonic bath with plaster solvent (e.g. FINO GIPS-EX U). Make always sure that the sealing surfaces of the mixing bowl and the mixing bowl pick-up are clean. Use only genuine spare parts.

4.5.1 Change of unit fuse

Pull plug from socket. The unit fuse is under the unit plug. Remove the fuse holder.

Check the fuse and replace it with the spare fuse from the fuse holder. Plug the unit plug back in and switch unit on. Replace the removed spare fuse with a new 250V/4A fuse.

4.5.2 Change of air filter

The air filter should be changed regularly depending on the frequency of use but at least once a year. Therefore remove the old air filter with a 13 mm socket key and put a new one in.

4.5.3 Replacement of rubber seal of the mixing bowl and the mixing bowl coupling

By using a flat screwdriver the rubber seals can be removed and can be replaced with new ones.

4.6 Troubleshooting

1. The unit does not switch on.

- Check circuit breaker of your house distribution.
- Pull the plug from the socket and check the unit fuse.

2. Material is not mixed properly.

- The filling level of the used mixing bowl is too low.

Use a smaller mixing bowl.

Observe the mixing ratio and mixing time indicated by the manufacturer.

3. Air bubbles in the mixed material

- Air filter is clogged.

See section 4.5.2.

4. The mixing bowl is not kept by the vacuum. The pressure gauge does not reach the green area.

- The lid of the mixing bowl is not clean. Therefore no or only a low vacuum can be created.

Clean the sealing surfaces of the mixing bowl and the rubber sealing of the mixing unit.

The rubber sealing of the mixing bowl or the mixing bowl holder is worn.

- Air filter is clogged.

Replace the air filter with a new one.

- Maximum filling level exceeded

Make sure that the maximum filling level of the mixing bowl in use is not exceeded and shortly mix the material manually with a spatula before mixing with the vacuum mixer.

5. Dried plaster or investment material in the mixing bowl.

- To remove smaller quantities of set plaster or investment material from the mixing bowl use e.g. FINO GIPS-EX U Plaster Solvent according to the user information in combination with an ultrasonic bath.

Should the malfunction not be overcome by the described measures please contact an authorised workshop or your supplier.

5. Storage

Store the unit in a dry and dust-free place. Storage temperature should be between +5 °C and +40 °C.

If not used for a longer period of time disconnect from mains.

6. Technical data

Dimensions: 15.5 x 20.5 x 35.0 cm
Voltage: 230 V, 50/60 Hz
Consumption: 100 W
Speed: 100-400 1/min
Max. vacuum: 650 mbar
Weight: 4,8 kg

7. Delivery forms

FINO MIXERY Vacuum Mixing Unit
Complete with mixing bowl 550 ml, power cable, spare air filter, mixing spatula and wall-mounting set.
each 10562

7.1 Accessories

Mixing bowls for FINO MIXERY		
	250 ml	10556
	550 ml	10557
	750 ml	10558
Alginate mixing bowl	230 ml	10559
Table stand	each	10560
Spare air filter	each	10561

8. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

9. Disposal in accordance with EU guideline 2002/96/EU

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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