SCHULER-DENTAL GmbH & Co.KG Eberhard-Finckh-Straße 39 89075 Ulm / Germany

Fon +49 (0) 731 9 27 72 - 0 Fax +49 (0) 731 9 27 72 - 49

info@schuler-dental.com www.schuler-dental.com

BEDIENUNGSANLEITUNG S-U-VIBRABABY, S-U-VIBRAKID

S-U-VIBRABABY Art.-Nr. 15 194 Geräte-Nr. G1 - S-U-VIBRAKID Art.-Nr. 15 195 Geräte-Nr. G4 -

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Dear customer,

By purchasing a product from SCHULER-DENTAL you habe decided on a high quality product.

Please read these operation instructions carefully before and during using this device for the first time to achieve good results from the very beginning.

Please pay particular attention to the notes highlighted with these singes.

Table of contents:

| Point | Subject | Page |
|-------|---|-------|
| 1. | Area and purpose of use | 4 |
| 2. | Starting up | 5 |
| 2.1 | S-U-VIBRABABY ,S-U-VIBRAKID | 5 |
| 3. | Service | 5 |
| 4. | Important notes | 5 |
| 5. | Technical data | 6 |
| 6,7 | Spare parts list | 6 |
| Add. | EU-Declaration of Conformity, | 7 - 9 |
| | Appliance certificate, warranty vertificate | |

= General advise for safety

= Advise for electrical safety

By the way:

Since we always strive to improve and further develop our products, we will be very pleased to receive your suggestions and ideas for improvements.

1. Area and purpose of use

These vibrators are used for the densification of mixed dental plaster, investment or similar materials which have not been set yet. e. g.: to pour out impressions, casting muffles and dental flasks for complete prosthetics.

2. Starting up

2.1 S-U-VIBRABABY . S-U-VIBRAKID

Vibrator, switch with 2 steps, for 1 flask ,for 2 flasks and to pour out impressions and casting muffles.

Switching on and changing of vibration intensity by means of a switch.

Centre switch position = "0" (OFF).

Switch position 1 = low vibration intensity.

Switch position 2 = high vibration intensity.

The good stability of the device is considerably improved by a smooth basis.

The four rubber suction buffers are most effective then.

3. Service

The rubber mats can be removed for cleaning from all types of devices.

A Please avoid any water to enter the inner part of the device's housing.

4. Important notes

A Place vibrator on a solid, not sloping basis. The device could move horizontally on a swinging or sloping basis and fall down.

A Please avoid any water to enter the inner part of the device's housing.

B Electrical appliances should generally not be left unsupervised for a longer period of time since the risk of a defect can never be completely excluded.

Therefore, electrical appliances should be switched off when the last person is leaving the room where the device is installed - provided the working procedure allows to do so.

B Repairs and adjustment works on the electrical parts of the appliance are to be carried out by an certified electrician only. Plug must be pulled out before opening the housing.

A Keep children away from appliance!

A In case of unauthorised constructional modifications or addition of components by staff not authorised by SCHULER-DENTAL, the EU-Declaration of Agreement becomes null and void and the manufacturer SCHULER -DENTAL does not assume any liability.

A In case of inappropriate use of the appliance SCHULER-DENTAL does not assume any liability.

5. Technical data

| | S-U-VIBRABABY | S-U-VIBRAKID |
|-----------------------|----------------|----------------|
| Mains voltage | 220-240 V/50Hz | 220-240 V/50Hz |
| Power consumption | 90 W | 120 W |
| Measurements | 120 x 120 mm | 120 x 210 mm |
| Heights | 110 mm | 140 mm |
| Oscillation frequency | 50 Hz | 50 Hz |
| Intensity | 2-steps | 2-steps |
| Weight | 2.8 kg | 4.5 kg |
| allowed room temp. | 10 - 40℃ | 10 - 40℃ |
| | [50 - 104°F] | [50 - 104°F] |
| L _{eq} *) | < 70 dB(A) | < 70 dB(A) |
| L _{max} **) | 78 dB(C) | 92 dB(C) |

^{*)} Equivalent continuous sound-pressure level, measured under the following conditions: Switch position coarse; pressure with maximum number of filled flasks; working time 50% of the total time of measuring.

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^{**)} Short-term peak level of the continuous sound-pressure level under the above mentioned conditions.

EG-Konformitätserklärung Déclaration de conformité CE Conférma di conformità di CE Declaration of CE Conformity EG-verklaring van overeenstemming Declaración de conformidad de CE EC-Единообразное пояснение

Hiermit erklären wir, daß Nous assurons que Confermiamo che Настоящим поясняется что SCHULER-DENTAL GmbH & Co. KG Eberhard-Finckh-Straße 39 D-89075 Ulm Germany

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Warranty certificate

This device was tested several times during production and before delivery thereof and was found without defect.

SCHULER-DENTAL commits themselves to removing free of charge defects, which are evidently due to material or production faults, within 12 months from date of purchase. This warranty applies to the first buyer only and cannot be transferred.

Repairs carried out on warranty do not result in a prolonged warranty period.

Consequential claims, particularly such as for replacement of parts damaged by external influence, are excluded from the warranty.

Excluded from the warranty are:

- damages caused by local conditions, such as erros in installation, fire, lightning strike, etc.
- damages caused by repair works carried out by persons not authorised by SCHULER-DENTAL.
- damages caused by not following the operation instructions, e. g., connection to the wrong mains voltage or type of current, and damages caused by negligence and misuse.
- batteries for battery powered appliances, as well as damages following the use of defect or old batteries.
- wearing parts.

Charges for transport and packing, as well as costs incurred for sending mechanics are at the client's expence. The warranty does not apply to damages caused during transport and are to be advised immediately.

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