FINOPASTE NG DUO
TOUCH/85/95

Silikonknetmasse

15156/15157/15157P/15098/15099/
15099P/15103/15104/15104P

REF

DEU Gebrauchsinformation

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REF

ENG User Information

Please read the entire document before using the product.

1. Intended purpose

Addition-crosslinked silicone putty
Silicone Putty for use in dentistry.

2. Indication

For fabrication of matrices, models for prosthetic repairs, control bites and to isolate plastic teeth in flasking.
Only for extraoral use!

2.1 Contraindication
Not for intraoral use!

Any use beyond that described in these instructions is considered to be improper use. The manufacturer shall not be held liable for any resulting damage.

3. Fields of application

3.1 General
For processing in the dental technology according to the intended use.

- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group
The target user groups for Silicone Putty are:

- Dental technicians

3.3 Cleaning
Silicone putty are products intended for single use. They do not require cleaning.

4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

No particular hazards worth mentioning.

5. Processing instructions

 Close container immediately after removal. Do not mix up the cover and dosage spoons!

 Do not wear latex gloves, latex impairs curing of the silicone modelling material. Vinyl gloves or PE-based gloves are recommended.

5.1 Dosing

Dose base and catalyst with dosage spoons at a ratio of 1:1 and spread on hand. Keep exactly to the mixing ratio to achieve correct curing times. Overdosing the catalyst does not accelerate the curing process.

5.2 Mixing

Knead both components well until a homogenous mixture (colour) has been achieved. Process the silicone modelling material.

5.3 Model fabrication

Casting the impression is possible after 60 min. All plasters of type III and IV which comply with ISO 6873 can be used. The impressions can be cast several times.
Recommended processing temperature: +18°C to +25°C / 64.4°F to 77°F.

6. Storage

 Store tightly closed at a temperature between 5–27°C / 41–80.6°F and 50% relative humidity.
 Avoid heat and sunlight!
 Shelf life: see label

7. Physical data

See table.

8. Delivery forms

The scope of delivery of each individual product consists of:
FINOPASTE NG DUO TOUCH
Silicone Putty

2 x 174 g	15157P
2 x 1500 g	15156
2 x 5000 g	15157

FINOPASTE NG DUO 85
Silicone Putty

2 x 181 g	15099P
2 x 1500 g	15098
2 x 5000 g	15099

FINOPASTE NG DUO 95
Silicone Putty

2 x 190 g	15104P
2 x 1500 g	15103
2 x 5000 g	15104

Silicone Putty are shown in the illustration:

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

 FINOGmbH · Mangelsfeld 18 · D-97708 Bad Bocklet · Tel. +49 97 08 90 94 20 · Fax +49 97 08 90 94 21
info@fino.com · www.fino.com

137214
08/25, Rev.-Nr.: 5.5

11.1 EWC waste code product
070216 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of plastics, synthetic rubber and man-made fibres; waste containing hazardous silicones; hazardous waste.

12. Symbols explanation

12.1 Manufacturing

REF

Item numbers

LOT

Lot code

Shelf life

Unit in the packaging

Manufacturer

12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

Notes

Important information

Not for intraoral use!

Heat resistance

Mixing ratio

Mixing time - kneading

Processing time

Setting time / curing time

Please read carefully before use
Keep for referral purposes

Do not reuse

12.3 Storage

Temperature limit

Keep away from direct sunlight

Protect against moisture

top - store upright

Caution fragile

	FINOPASTE NG DUO 95	FINOPASTE NG DUO 85	FINOPASTE NG DUO TOUCH	
Physical data				
Consistency	kneadable	kneadable	kneadable	
Colour base/catalyst	violet/light grey	beige/light grey	dark grey/light grey	
Mixing time	30 s	30 s	30 s	
Processing time incl. mixing time	1 min 20 s	1 min 20 s	1 min 30 s	
Total setting time	6-7 min	6-7 min	6-7 min	
Final hardness Shore A	approx. 95	approx. 85	approx. 80	
Flavouring	blueberry	creamy vanilla	neutral	
Boil-proof	✓	✓	✓	
Suitable for microwave ovens	✓	✓	✓	

FINOPASTE NG DUO
TOUCH/85/95
silicone malléable
15156/15157/15157P/15098/15099/15099P/15103/15104/15104P

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