

# **blue eco lab putty STONE**

**Putty for laboratories, silicone based,  
addition curing - 96 Shore A**

**Instructions for use:** Remove impression material A (white) and B (blue) using measuring scoops. Mix together (mixing ratio 1:1, approx.30 sec.) until a homogeneous mix or uniform colouring is achieved.

**Caution:** *For extraoral use only.*  
Avoid mutual contact of components A + B before mixing. Do not interchange measuring scoops and lids. Do not remove material from container by hand. Do not use latex-gloves.

**Storing:** Keep tightly closed at temperatures not exceeding 25°C/77°F.

**Mixing time:** approx. 30 sec. \*  
**Working time:** 1min. 30 sec. \*  
**Setting time:** approx. 4 min. 30 sec. \*

\* from beginning of mixing; increased temperatures accelerate, decreased temperatures retard the setting time.

Made in Germany

**DETAX**

GmbH & Co. KG  
Postfach 10 02 25 • 76256 Ettlingen/Germany  
Telefon: 07243/510-0 • Fax: 07243/510-100  
www.detax.de • post@detax.de