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**coltène®**

## Lab-Putty

### Definition

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# Lab-Putty

## Definition

Coltène® Lab-Putty is an elastomeric putty for use in the dental laboratory.

## Material type

Polysiloxane, condensation-type elastomer.

## Indications

Coltène® Lab-Putty is exclusively designed for extraoral application.

It has been conceived for the manufacture of:

- Matrix  
Repro, general occlusion and functional pattern (gfp), matrix as positioner for clasps, denture teeth (i.e. partial dentures, temporary partials, orthodontic retainers etc.), matrix for making gingival masks Coltène® Gi-Mask.
- Repairmodels  
(lower dentures with deep undercuts)
- Block-outs
- Hard palate relief stamp (mould)
- Labial/buccal positioner matrix

## Contra-indications

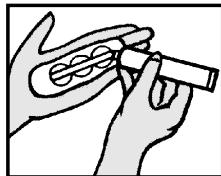
Provided that the product is used as directed, there are no known contra-indications.

## Side effects / Interactions

Polysiloxanes have a very good biological compatibility.

Up to now no harmful reactions or secondary effects on laboratory staff are known.


## Dosage

Measure out Coltène® Lab-Putty Base with the measuring scoop (take off excess). Spread measured out quantity on the palm of the hand. For each scoop Base press the scoop-rim once into the mass. Dose one strand-length of Coltène® Lab-Putty Activator per circle diameter.

**Close tube immediately after use.**
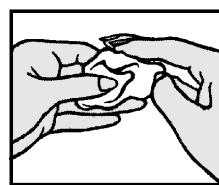
## Mixing

Fold material and knead with fingertips for 30–45 s until a homogeneous, pink-colored mixture is produced.

## Caution:

Federal law restricts this device to sale by or on the order of a dentist.

## Coltène/Whaledent AG (Manufacturer)

Feldwiesenstrasse 20  
9450 Altstätten/Switzerland  
Tel. +41 (0)71 757 53 00  
Fax +41 (0)71 757 53 01  
office@coltenewhaledent.ch

## For MSDS see

[www.coltenewhaledent.com/msds.htm](http://www.coltenewhaledent.com/msds.htm)

## Setting time

Min. 5–7 min at room temperature.

Intensive mixing, high temperatures or over-dosage of the activator speed up the setting.

Low temperatures and under-dosage of the activator slow down the setting.

Max. over-dosage / under-dosage: 20 %

## Shelf life and storage

Coltène® Lab-Putty fulfills the intended purpose at least until the expiry date, which is displayed on the containers, when stored at 15–23 °C / 59–73 °F and 50 % relative air humidity.

## Marking

The expiry date and **LOT** number are shown on the package and cartridge.

## Technical data

<u>Mixing time:</u>	30–45 s
<u>Total working time:</u>	ca. 90 s
<u>Setting time:</u>	5–7 min
<u>Shore A hardness:</u>	63–70

## Mixing conditions

The stated working time is based on measurements at 23 °C / 73 °F room temperature and 50 % relative humidity.

## Instructions for use date

June 2004

Only supplied to dentists and dental laboratories or upon their instructions.

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