**REF** 19014

**DEU** Gebrauchsinformation

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REF 19014

### ENG User Information

Please read the entire document before using the separating agent.

#### 1. Intended purpose

Separating agent for polyurethane-based modeling material.

#### 2. Indication

For separating polyether impression taking and duplicating materials.

#### 2.1 Contraindication

Not known.  
In case of allergic reactions to product components, sensitive individuals cannot be excluded.

#### 3. Fields of application

##### 3.1 General

Only for processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

##### 3.2 Target group

The target user groups for Separating agent are:

- Dentists
- Dental technicians

The patient target group for Separating agent includes patients receiving as part of dental treatment.

#### 3.3 Cleaning

Separating agent are products intended for single use. They do not require cleaning.

Separating agent are not intended for disinfection.

#### 4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

Danger

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H336 May cause drowsiness or dizziness.

H304 May be fatal if swallowed and enters airways.

H411 Toxic to aquatic life with long lasting effects.

##### 4.1 Prevention

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

##### 4.2 Response

P310 Immediately call a POISON CENTER or doctor/physician.

P301+P310 IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician.

P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P405 Store locked up.

#### 5. Processing instructions

Spray insulating agent on the surface to be insulated with the pump atomiser. Leave to air dry for at least 7 min. Proceed processing model material as usual.

#### 6. Storage

P234 Keep only in original container.

P411 Storage temperature from 10 to °C 25.

P402+P404 Store in a dry place. Store in a closed container.

P403+P235 Store in a well-ventilated place. Keep cool.

 Protect against frost.

#### 7. Physical data

Wax in light oil (contains solvent).

#### 8. Delivery forms

Packaging units for Separating agent are:

- Separating Agent
- User Information

The scope of delivery of each individual product consists of:

FINOPONT SEPARATOR Separating Agent  
100 ml 19014

Separating agent are shown in the illustration:

Separating agent are supplied without accessories.

#### 9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

#### 10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

#### 11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

##### 11.1 EWC waste code product

070104 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the manufacture, formulation, supply and use (MFSU) of basic organic chemicals; other organic solvents, washing liquids and mother liquors; hazardous waste.

#### 12. Symbols explanation

##### 12.1 Manufacturing

 Item numbers

 Lot code

 Shelf life

 non-sterile

 Unit in the packaging

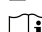
 Manufacturer

#### 12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

 Notes

 Important information

 Please read carefully before use  
Keep for referral purposes

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