FINO TORCH

Laborbrenner

REF 20146/20147

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REF 20146/20147

ENG Instructions for use

Please read carefully before use
Keep for referral purposes

Only to be used by qualified
personnel!

1. Intended purpose

Refillable soldering and welding torch for use in dentistry.

With piezo ignition and safety switch.

2. Indication

Suitable for fine, precise soldering and welding work up to 1300°C. Adjustable flame size.

2.1 Contraindication

Not known.

3. Fields of application

Only for processing in the dental laboratory according to the intended use.

4. Safety precautions

Do not throw or open burner by force.

When igniting the burner, always keep a sufficient safety distance from your body.

5. Operation

5.1 Commissioning

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

The unit must only be used for the intended purpose described in the scope/field of application.

5.2 Handling

Fill the laboratory burner with gas as described in Section 5.5.

Slide the red safety switch down and press the red ignition button. The piezoelectronic ignition ignites the escaping gas and generates an adjustable flame.

The flame is extinguished as soon as the button is released.

5.3 Continuous operation

To activate continuous operation, move the red controller on the left side of the burner from "OFF" to "CONTINUOUS" while keeping the ignition button pressed.

The ignition button can be released.

To end the burning process, move the red controller back to "OFF".

5.4 Flame control

To control the flame, move the gas regulator on the right side of the burner to the right ("+" = increase flame) and to the left ("- " = decrease flame).

5.5 Refilling gas

Switch off the burner and make sure that the flame is extinguished and that the device has cooled down. Hold the burner upside down so that the refill valve on the bottom of the burner points upwards.

Also hold the butane gas cartridge (e.g. FINOGAS burner gas, Art. 20103) upside down and insert the nozzle of the butane gas cartridge into the refill valve of the burner. Refilling must be stopped as soon as gas overflows from the burner tank. After filling, wait a few minutes to allow the refilled gas to stabilise. Only then may the device be ignited again.

6. Storage

P410+P412 Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.

P403+P235 Store in a well-ventilated place. Keep cool.

7. Technical data

Dimensions 12.8 x 6.5 x 15.6 mm; temperature max. 1300°C; burning time ca. 75 min.; filling volume max. 40 ml; filling gas butane; flame size adjustable; weight 183 g.

8. Delivery forms

FINO TORCH Laboratory burner
Plastic handle 1 piece 20146
Aluminium handle 1 piece 20147

9. Guarantee

A condition for guarantee claims is the availability of the original sales invoice from the dental retailer.

Our products are subject to continuous further development. We therefore retain the right to changes in design and composition. Of course we guarantee the perfect quality of our products.

! Important

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document. Performances under guarantee do not extend the term of guarantee.

10. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

10.1 EWC waste code product 100810 WASTES FROM THERMAL PROCESSES; wastes from other non-ferrous thermal metallurgy; dross and skimmings that are flammable or emit, upon contact with water, flammable gases in hazardous quantities; hazardous waste

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