REF 40505/40505P

DEU Gebrauchsinformation

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REF 40505/40505P

## ENG User Information

Please read the entire document before using the investment materials.

### 1. Intended purpose

Volume-stable, plaster-bonded investment material for soldering work.

### 2. Indication

For soldering abutments to partial dentures. Also suitable for repairs.

#### 2.1 Contraindication

Not known.

In case of allergic reactions to product components, sensitive individuals cannot be excluded.

### 3. Fields of application

#### 3.1 General

Only for processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

#### 3.2 Target group

The target user groups for Investment materials are:

- Dentists
- Dental technicians

The patient target group for Investment materials includes patients receiving as part of dental treatment.

### 3.3 Cleaning

Investment materials are products intended for single use. They do not require cleaning.

Investment materials are not intended for disinfection.

### 4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

No particular hazards worth mentioning.

Further safety-relevant data is given in the safety data sheet.

### 5. Processing instructions

Note:

Mix the investment material thoroughly after prolonged storage.

#### 5.1 Mixing

Mix the powder and liquid in a clean mixing bowl with a spatula:

Mixing time with vacuum: 30 sec.

Mixing time manually: 60 sec.

#### 5.2 Processing

Fix the solder in the still soft investment material. Trim the solder block after appropriate setting time.

To avoid heat loss, the solder block should be created as small as possible and the solder joint should remain free.

The solder block can be removed easily under running water with a brush.

### 6. Storage

P234 Keep only in original container.

P403+P235 Store in a well-ventilated place. Keep cool.

### 7. Physical data

Processing temperature 21 °C; mixing ratio 100 g : 23 ml; setting time 6-10 min.; temperature: max. 1100 °C; colour: brown.

### 8. Delivery forms

Packaging units for Investment materials are:

- Soldering investment material
- User Information

The scope of delivery of each individual product consists of:

FINOTHERM Soldering investment material	4500 g	40505
sample pack	1000 g	40505P

Soldering investment material are shown in the illustration:

Investment materials are supplied without accessories.

### 9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

### 10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

### 11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

#### 11.1 EWC waste code product

180107 WASTES FROM HUMAN OR ANIMAL HEALTH CARE AND/OR RELATED RESEARCH (EXCEPT KITCHEN AND RESTAURANT WASTES NOT ARISING FROM IMMEDIATE HEALTH CARE); wastes from natal care, diagnosis, treatment or prevention of disease in humans; chemicals other than those mentioned in 18 01 06.

### 12. Symbols explanation

#### 12.1 Manufacturing

- Item numbers
- Lot code
- Shelf life
- non-sterile
- Unit in the packaging
- Manufacturer

#### 12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

- Notes
- Important information

Please read carefully before use  
Keep for referral purposes

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