REF 41593

DEU Gebrauchsinformation

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REF 41593

ENG User Information

Please read carefully before use
Keep for referral purposes

1. Intended purpose

For melting and casting dental precious metal alloys.

2. Indication

Only for processing in the dental technology according to the intended use.

2.1 Contraindication
Not known.

3. Fields of application

Only use the product for the application described in the intended use.

4. Safety precautions

No particular hazards worth mentioning.

5. Processing instructions

Heat the casting device to the required temperature and insert the crucible. Avoid long heating times.

If present, close the lid of the casting device. As soon as the graphite crucible is red-hot, pour the alloy into the crucible according to the manufacturer's instructions, melt and cast as quickly as possible.

Do not interrupt the casting process if the melt has already turned liquid! There is a risk that the crucible will melt through.

Once the casting process is finished, allow the graphite crucible cool down in the switched off device.

To accelerate the cooling process, place the crucible in a sand bath.

Under no circumstances cool down with cold water, as fine cracks can form in the graphite body. Never leave the graphite crucible unnecessarily in the heated casting device, as each additional dwell time in the heated device promotes burn-off.

6. Storage

P402 Store in a dry place.

7. Physical data

No further data.

8. Delivery forms

FINO FLOWCAST Graphite inserts
5 pieces 41593

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

10.1 EWC waste code product
170107 CONSTRUCTION AND DEMOLITION WASTES (INCLUDING EXCAVATED SOIL FROM CONTAMINATED SITES); concrete, bricks, tiles and ceramics; mixtures of concrete, bricks, tiles and ceramics other than those mentioned in 17 01 06.

11. Symbols explanation

11.1 Manufacturing

- Item numbers
- Lot code
- Shelf life
- non-sterile
- Unit in the packaging
- Manufacturer
- Date of manufacture

11.2 Application

- Please read carefully before use
Keep for referral purposes
- Only to be used by qualified personnel!
- Do not reuse
- Biohazard

11.3 Storage

- Temperature limit
- Keep away from direct sunlight
- Protect against moisture
- Do not use damaged packaging
- top - store upright
- Caution fragile

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