FINO WAX-EX
Wachsausbrühmittel

REF 46075/46076

DEU Gebrauchsinformation

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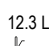
12.3 Lagerung

 Temperaturbegrenzung

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Empty box for additional information.

Empty box for additional information.

Empty box for additional information.

 **FINO WAX-EX**
Wax Scalding Agent

REF 46075/46076

ENG User Information

Please read the entire document before using the Wax scalding agent.

1. Intended purpose
Additive concentrate for better dissolving of wax for use in wax extraction units.

2. Indication
Suitable for all wax types and all extraction units.

2.1 Contraindication
Not known.

In case of allergic reactions to product components, sensitive individuals cannot be excluded.

3. Fields of application
3.1 General
Only for processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group
The target user groups for Wax scalding agent are:

- Dentists
- Dental technicians

The patient target group for Wax scalding agent includes patients receiving as part of dental treatment.

3.3 Cleaning
Wax scalding agent are products intended for single use. They do not require cleaning.
Wax scalding agent are not intended for disinfection.

4. Safety precautions
The national legal regulations must be observed during use, in particular:

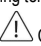
- the applicable health and safety regulations
- the applicable accident prevention measures

 Attention
H319 Causes serious eye irritation.

4.1 Prevention
P264 Wash hands thoroughly after handling.
P280 Wear eye protection/face protection.

4.2 Response
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P337+P313 If eye irritation persists: Get medical advice/attention.

5. Processing instructions
Mix Wax Scalding Agent at a ratio of 50 ml concentrate : 5 l cold water. Heat the liquid to the desired temperature in an extraction unit and place the objects.
Do not switch on the spray function until the working temperature has been reached!

 Caution:
Do not heat over 100 °C!

6. Storage
P233 Keep container tightly closed.
P234 Keep only in original container.
 Avoid heat and sunlight!

 Protect against frost.

7. Physical data
Low viscosity, non-foaming, pH value 7.8, virtually water-insoluble, thermal decomposition > 110 °C.

7.1 Composition
Distilled water, sodium benzoate, fatty alcohol, terpene.

8. Delivery forms
Packaging units for Wax scalding agent are:

- Wax Scalding Agent
- User Information

The scope of delivery of each individual product consists of:
FINO WAX-EX Wax Scalding Agent
1000 ml 46075
5000 ml 46076

Wax Scalding Agent are shown in the illustration:

Wax scalding agent are supplied without accessories.

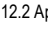
9. Guarantee
Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability
All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal
P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product
070701 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fine chemicals and chemical products not otherwise specified; aqueous washing liquids and mother liquors; hazardous waste

12. Symbols explanation
12.1 Manufacturing

	Item numbers
	Lot code
	Shelf life
	non-sterile
	Unit in the packaging
	Manufacturer

12.2 Application

 General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

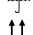
 Notes

 Important information

 Please read carefully before use
Keep for referral purposes

 Do not reuse

12.3 Storage

-  Temperature limit
-  Keep away from direct sunlight
-  Protect against moisture
-  top - store upright
-  Caution fragile

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