FINO HOTPLATE

Heizplatte

REF 47608

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REF 47608

ENG Operating Instructions

Please read the entire document before using the Hotplate.

1. Intended purpose

Hotplate for use in dentistry.

2. Indication

For heating pressure pots for the polymerisation of dentures etc. Infinitely adjustable.

2.1 Contraindication

Do not use vessels made of plastic, inferior enamel, porcelain or aluminium foil.

Do not use pots with a too small a diameter and a concave or convex bottom.

At a diameter of approx. 20 cm, the inward curvature must not exceed 1 mm.

3. Fields of application

For processing in the dental technology according to the intended use.

4. Safety precautions

⚠ There is a risk of burns from heat!

5. Operation

5.1 Commissioning

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

Place the device on a heat-resistant, stable and level work surface.

Before first use, heat the hotplate without pot for approx. 5 minutes at the highest setting.

This gives the plate coating its optimum protective effect.

Smoke and odours may develop briefly, but these are not harmful.

5.2 Commissioning

Caution:

Never operate the device unattended or near flammable liquids or gases!

The space above the device must be kept free and ensure unobstructed air circulation!

Do not allow connecting cables to come into contact with the heated hotplate!

Only operate the device under supervision and out of reach of children!

The control lamp lights up during the heating up period. Once the temperature is reached, it is extinguished.

In switch position 3 the water temperature is approx. 45°C (1 litre).

5.3 Service and maintenance

Switch unit off and unplug.

Clean the hotplate with detergent and a soft sponge, do not use abrasive cleaning agents.

Never immerse the device in water. Make sure that no steam or water enters the housing from underneath.

6. Storage

Store the device at room temperature protected against moisture.

7. Technical data

Rating: 230 V, 50 Hz; power: 2000 W; dimensions: 25.0 x 28.0 x 7.5 cm; hotplate: Ø 18 cm; weight: 2.8 kg; colour: white

8. Delivery forms

FINO HOTPLATE Hotplate

1 piece 47608

Hotplate are shown in the illustration:

9. Guarantee

A condition for guarantee claims is the availability of the original sales invoice from the dental retailer. Our products are subject to continuous further development. We therefore retain the right to changes in design and composition. Of course we guarantee the perfect quality of our products.

! Important:

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document.

Performances under guarantee do not extend the term of guarantee.

10. Disposal in accordance with EU guideline 2002/96/EU

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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FINO GmbH · Mangelsfeld 18 · D-97708 Bad Bocklet · Tel. +49 97 08 90 94 20 · Fax +49 97 08 90 94 21