IP Clean 2 **Dampfstrahler**

Bedienungsanleitung / Istruzioni / Instruction
Notice d'Instructions /Instrucciones para el Uso

IP Division GmbH Pfarrstr.3 D-85778 Haimhausen
Tel:+49-(0)-8133-444951 Fax:+49-(0)-8133-444953
E-Mail:info@ipdent.de www.ipdent.de

ANMERKUNGEN

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IP Division

“ EC “Declaration of Conformity

The company IP Dental Division
Pfarrstr.3 85778 Haimhausen
declares the machine
Steam Generator Clean,Clean 2.Clean2i with Steam gun
Year of construction:2006

has been manufactured in conformity with the following European Directives:
Machine: 89/392 Cee.Low Voltage: 73/23 Cee amendments by 93/68 CEE.
Électromagnetic Compatibility: 89/336 CEE and further amendments.

Haimhausen, 2006

IP Division

Notices

The pictures in the present handout ,descriptions and qualities are not binding ,apart from the main qualities the producer reserves the privilege in every time to make changes in technical or commercial managements.

The steamer is only for the use in dentaltechnical,the dentistry projected and realized.

Other applications are from the producer not noted down. He will deny every responsibility concerning to all damages which will arise from careless use of the tool.

Changes ,unauthorized interventions (only in a small degree) and using from no original Spare parts will exempt the producer from from every responsibility and you will have no Right for gurananty.

The tool should only used from adults and educated personal.

Security rules

- please make sure after the unpacking,that you have got the completely tool and the accessories.
- Before connecting the tool please make sure, that the datas of the model label are the same than the current power
- Connect the powercable, how described in the European rules or the rules in your country in which it will be used on to the wall socket.The collector must absolutely have earth to ground.

Attention please!

The ground earth is a commanded guarantee by law and must put on very carefully by a qualified technician engineer.

- Don t use the tool ,if there is a defective cable or a defective plug.
- Please only use the tool in rooms,where are no environmental influences like rain or snow.

Attention please!

You only should use normal water in connection with waterlimer or distilled water.

Other liquids ,which developes poisonous gases by become heated you never should use.

Attention please!

Never open the lock as long as the boiler is under pressure.Either at the boiler is not under pressure you should open the lock very carefully. First unscrew the lock easy for let of the steam completely,then you can quite open the screw-thread.

Service instructions

- please put the plug off and fill it with the help of a funnel with the minimum from 1 utmost 2 Liter with water.
- Please lock the screw lock very carefully and test it
- Connect the tool with the system plug
- Switch on both switch keys (1) and test whether the control lamp is lightning.
- Wait about 10-15 minutes up to the boiler is under pressure and so the boiler control lamp will go out.
- Trough pressing of the foot pedal (3) you can use the steamer.

Attention please!

Please always empty the boiler before use. Therefore will the rest water from former use flow away and the stored limer arears on the ground will be removed.

Filling up of the boiler

Attention! Never open the lock during the boiler is under pressure.

- When the control lamp (2) shows,that the water is used up,please switch off the main switch
- First unscrew the lock easylike and let the steam completeley leave,then open the screw completely.
- After removing the lock,please wait before filling up the boiler again ca. 10/15 minutes

Caution! Very important: Before the next use, please let the boiler cool down, therefore you won t cause a temperature shock or break.

- Now fill 1-2 Liter normal water into the boiler
- Put on the lock and make sure, that it will lock well
- Switch on ,wait 5-10 minutes up to the boiler will come under pressure.

Maintenance

- please empty the boiler after each use for removing the rest water and the originated uncleanliness .
 - You don t need more maintenance for the Clean 2
 - For Cleaning only use a light damp rag without alcohol or solution
- Caution!Only empty when the powercable is put off.

Guarantee

- 12 month beginning with the buying date
- The boiler made of AISI steel 5 years

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Breakdowns	Possible causes	remedys
No steam	The key is defect	Put off the plug and untie the screws of the pistol and clean the contacts. If you cannot execute the breakdown the key will be replaced.
	The magnet valve closes	replace
	The sprayer nozzle is calcined	Unscrew the nozzle and put it in diluted vinaigre or other current way IP limer
Steam comes out of the cover	The cover is not screwed in the right way	Screw the cover in the right way
	The inner seal is defect	replace the seal
	The pressure guard is defect	replace the pressure guard
The tool doesn't heat	The heating is defect	Call the technical service
The tool heats to slowly	The boiler is blocked with deposit	Inlime the boiler
The control light of the boiler doesn't stop		empty the boiler and fill up again with 1 ½ l. the problem cannot be solve. Refer the technical service