FINOCARAT PLUS I/II

Diamantpolierpaste

REF 68018/68019

DEU Gebrauchsinformation

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REF 68018/68019

ENG Instructions for use

Please read carefully before use
Keep for referral purposes

1. Intended purpose

Diamond pastes are made from high-quality natural diamonds for the creation of smooth surfaces.

2. Indication

Diamond polishing pastes with different diamond concentration.

2.1 Contraindication

Do not use if incompatibilities or allergies to constituents have been confirmed.

3. Fields of application

Due to the different diamond concentrations the indication for all dental materials is covered.

4. Safety precautions

No dangers worthy of special mention.

5. Processing instructions

FINOCARAT PLUS I and II contain natural diamonds in a very high concentration. FINOCARAT PLUS I and II can be used very economically. One „drop“ is sufficient for a multi-unit bridge. The diamond pastes are used undiluted.

Before polishing the objects are thoroughly cleaned and dried.

Subsequently the surfaces are polished under low pressure at a speed of 10.000 – 15.000 rpm using wheel or cup brushes or felt discs. The surfaces are processed until they show a nice high-shine. The crowns and bridges can now be simply cleaned with a steam cleaner.

6. Storage

P233 Store tightly closed.
P234 Keep only in original container.

7. Physical data

Grain size 2 µm; density 0,12 g/cm³, pH-value 4-7

8. Delivery forms

FINOCARAT PLUS I diamond paste
5 g 68018
FINOCARAT PLUS II diamond paste
5 g 68019

9. Guarantee

Thank you for choosing a quality product from Dentaurum. It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dt-shop.com, even though you may frequently use the same product.

9.1 Quality

FINO ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences.

The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as we, FINO, have no influence on the processing on site.

10. Disposal

Please adhere strictly to the rules that apply for your region, bearing in mind the details outlined in the safety data sheets.

11. Symbols explanation

11.1 Manufacturing

REF Item numbers

LOT Diamantpolierpaste code

Shelf life

non-sterile

Unit in the packaging

Manufacturer

Date of manufacture

11.2 Application

Rx only Only to be used by qualified personnel!

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