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WEEE-Reg.-Nr.

DE 75542610

Please read the entire document before using the product.

1. Intended purpose

Headband Loupe for use in dentistry.

2. Indication

Headband magnifier with various magnification options and individual headband adjustment.

2.1 Contraindication

Intended for use only in the specific area of application of the dental laboratory.

Any use beyond that described in these instructions is considered to be improper use. The manufacturer shall not be held liable for any resulting damage.

3. Fields of application

For processing in the dental technology according to the intended use.

4. Safety precautions

⚠ Caution! Failure to comply with the following safety precautions can lead to serious injury of the operator(s) and/or other persons.

⚠ Risk of fire if handled or stored inappropriately due to burning glass effect of the lenses. Never subject optical lenses to direct sunlight without a cover.

⚠ Risk of dazzling and injury!
Do not look into the sun with optical devices or other bright light sources.

5. Operation

5.1 Commissioning

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

5.2 Preparation of light source

Pull LED module to the stop and release by pressing the locking mechanism (see Fig. 1). Remove LED module. Open closing cover. Insert 3 batteries type AAA (see Fig. 2). The LED module can also be used separately (see Fig. 3). (Batteries are not included).

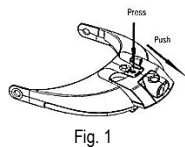
Fig. 1

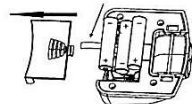
Fig. 2

Fig. 3

5.3 Inserting/removing the magnifying glasses

Insertion: align magnifying glasses with their convex side facing away from the eye and insert in holder. Serial connection is possible. (see Fig. 4). Removal: pull magnifying glasses **parallel** from the holder to avoid damaging them.

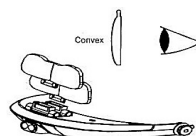
Fig. 4

5.4 Magnification options

Magnification	Working distance
1.0x	approx. 300-350 mm
1.5x	approx. 240-260 mm
2.0x	approx. 215-235 mm
2.5x	approx. 205-225 mm
3.0x	approx. 160-180 mm
3.5x	approx. 135-145 mm
4.0x	approx. 125-135 mm
4.5x	approx. 110-130 mm
5.0x	approx. 100-120 mm
5.5x	approx. 80-95 mm
6.0x	approx. 60-85 mm

5.5 Service

Clean with a soft cotton wool or linen cloth. Only wipe gently over the lens surface. Rinse with lukewarm water in case of stronger contamination. Do not use alcohol-containing or organic solvents!

6. Storage

Store the device at room temperature protected against moisture.

7. Delivery forms

The scope of delivery of each individual product consists of:

FINOZOOM Headband Loupe
with removeable LED module

5 magnifying glasses
(1.0x, 1.5x, 2.0x, 2.5x, 3.5x)

1 piece 80016

Headband Loupe are shown in the illustration:

8. Guarantee

A condition for guarantee claims is the availability of the original sales invoice from the dental retailer.

Our products are subject to continuous further development. We therefore retain the right to changes in design and composition. Of course we guarantee the perfect quality of our products.

! Important:

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document. Performances under guarantee do not extend the term of guarantee.

9. Disposal in accordance with EU guideline 2002/96/EU

⚠ This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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