HEINE QUALITY
MADE IN GERMANY

∴ HEINE C 2.3 Binocular Loupe

 **HEINE Optotechnik GmbH & Co. KG**
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com
MED 2199 2021-01-27

V-200.00.024

HEINE C 2.3 Binocular Loupe

DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
SUOMI	24
PORTUGUÊS	27

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Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Intended Use

Use only as a magnifier for examination and operations.

OPERATE

Adjustment

To get the best results from this loupe, the optics must be adjusted exactly. This is done as follows:

Getting a single clear image

- Unscrew both set screws so that the eyepieces can be slid in or out.
- Put on the loupe and look at a well lit three-dimensional object at working distance.
- Adjust one eyepiece after the other until you see a round image with no shadows and a focused object in each.
- Screw in both set screws softly without moving the eyepieces.
- Finally screw in both set screws firmly.

Spectacle Frame

The frame can be adapted to your needs by gently bending the nasal supports and adjusting the temple length to ensure that it is comfortable.

HYGIENIC CLEANING / PROCESSING

Protection, Maintenance and Cleaning/Disinfection

The optics of the loupes can be easily damaged and should be protected against scratching, usually caused by dust or debris. Please attach the detachable protective lenses when using the loupes. When not in use, store the loupes in the protective case provided.

Cleaning

Before cleaning, remove the protective lenses. For cleaning or removing fingerprints from the optics use a micro-fiber cloth. The housing can be cleaned by wiping with a damp cloth. Use pure water or a pH-neutral or mild soapy / alkaline solution.

Caution: Do not rinse the loupes under running water. Do not immerse the loupes in liquids. Avoid penetration of liquids into the optics.

Caution: Hard and / or metallic brushes can cause scratching and damage to the optics.

Caution: Do not use any abrasive cleaners or aggressive detergents for cleaning.

Disinfection

Only disinfection by wiping with a damp cloth is recommended. For disinfection use only detergents which are recommended for use on medical devices made of plastic and glass. Isopropyl alcohol should only be used at a concentration below 70%.

Maintenance

This product does not require regular maintenance.

Service

This product does not require regular service.

Disposal

Please observe the relevant state-specific disposal regulations.

Technical specification

Magnification*:	2.3x
Working distance*:	340 mm and 450 mm
Pupillary distance:	50 – 74 mm
Weight:	46 g
Frame:	Metal, rhodium-plated

*: For non-corrected refractive errors working distance will differ from the above values.

As a general rule the working distance will decrease with near-sightedness and increase with far-sightedness.

General Notes and Warnings

 Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.

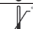
- Do not look into the sun or into a powerful light source with this loupe.

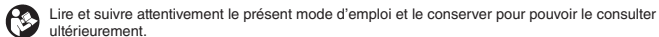
This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

 This product is a precise optical instrument. Please handle this product with care and store it in a clean place. The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.
	Catalogue- or reference number
	Serial number
	Manufacturer
	Date of manufacture
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Please read and follow the instructions for use and keep them for future reference (background color blue; foreground color white)
	Fragile, handle with care!
	Keep dry
	Unique Device Identification

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