**FINO**  
LED-Gelenkleuchte

REF 80895/80896

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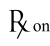
WEEE-Reg.-Nr.

DE 75542610

 Please read carefully before use  
Keep for referral purposes

 Only to be used by qualified personnel!

### 1. Fields of application

For workplaces with very high light requirements. With variably adjustable dim and memory function. With 3D hinge at the light head.

### 2. Safety precautions

Repairs of the unit must only be done by DT&SHOP or an authorised workshop. The user is liable for damages that result from arbitrary repairs. The terms of guarantee and delivery of DT&SHOP are unrestrictedly valid for this unit.

Operate the light only in dry rooms (rating IP 24). Avoid direct sunlight to the lens. Fire hazard due to burning glass effect. Cover lens when not in use. Do not operate the light without diffuser cover. Avoid frequent on/off switching within short intervals because this will reduce the service-life of the light.

Always disconnect the lamp from the mains before opening.

### 3. Processing instructions

#### 3.1 Installation

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

Attach the light in a way that guarantees the maximum illumination of the workplace.

Note:

The long light arm creates forces that must be balanced.

There are three attachment options:

1) Attach the screw-on clamp directly to the work top with four suitable screws. The screws must be directly drilled into the work top and will leave drill holes.

2) Use of the table clamp available as accessory (item-no. 85058): Connect the screw-on socket with the table clamp using the four included screws. Clamp the table clamp to the edge of the work top. The work top is not damaged.

3) Use of the tabletop stand (item-no. 85113) available as accessory. Detach screw-on clamp and insert the linkage of the light directly into the opening of the table stand. The work top is not damaged.

#### 3.2 Operation

The on/off switch is located on the joint arm. Pressing the button briefly will switch the lamp on or off. The light can be brightened or dimmed by pressing the button and holding. The light strength changes after the button is released. The lamp has a memory function. It switches on when set to the lowest light setting before the "off" setting.

Bring the lamp into the required position by adjusting the joints while the wing nuts are closed. The wing nuts are only used for readjustment. The socket head screw opposite the butterfly screw cannot be adjusted subsequently due to the adhesive. The lamp head can be adjusted in all directions thanks to the integrated ball joint.

„Soft-Light“-function:

An additional diffusor foil is inserted behind the bevelled glass pane. Turning of the foil reduces the reflection glare on high-shine surfaces considerably. The light distribution becomes softer but the illumination level is reduced by approx. 14 %.

To select the „Soft-Light“-function switch the light off and let cool for a couple of minutes. Remove the front screw of the side plate without on/off switch. Loosen the back screw a bit and rotate the head part away. Firstly, pull the cover plate carefully out and then the thin diffusor foil, turn it 180° and re-insert it (rough side up). Never touch the inner reflector with bare fingers because the natural skin fat impairs the reflector properties. Reassemble the light in reversed order.

#### 3.3 Service and maintenance

Clean the light with a dry cloth and a mild cleaning agent. Do not use abrasive cleaning agents! Clean the particularly sensitive reflectors of the light only with a dry micro-fibre cloth.

The lamp features highly efficient LED modules with an average service life of approx. 50,000 hours. If one or more of the LED modules should cease to operate, return the lamp as special thermal conductive foils are used during assembly and the LED modules must be replaced in an ESD protected area (EPA).

#### 4. Storage

Place unit in a dry and dust-free place.

Storage temperature: +5 up to + 40 °C.

If unit is not used for a longer period disconnect from mains.

### 5. Technical data

#### 5.1 Short version

Light head 350 x 99 x 35 mm; lower hinged arm 445 mm; upper hinged arm 360 mm; 224-240 V, 50/60 Hz; consumption 15 W; 24 LEDs, colour rendering index Ra 95; colour temperature 5700 K; illumination level max. 3500 lx (at 400 mm measured distance); power cable 2.0 m.

#### 5.2 Long version

Light head 586 x 99 x 35 mm; lower hinged arm 445 mm; upper hinged arm 360 mm; 224-240 V, 50/60 Hz; consumption 27 W; 48 LEDs, colour rendering index Ra 95; colour temperature 5700 K; illumination level max. 4700 lx (at 400 mm measured distance); power cable 2.0 m.

### 6. Delivery forms

FINO LED hinged light complete with illuminant and screw-on clamp

Long version	1 piece	80895
Short version	1 piece	80896

### 7. Accessories

Bench clamp	1 piece	85058
Tabletop stand	1 piece	85113
Screw-on clamp	1 piece	85059

### 8. Guarantee

A condition for guarantee claims is the availability of the original sales invoice from the dental retailer.

Our products are subject to continuous further development. We therefore retain the right to changes in design and composition. Of course we guarantee the perfect quality of our products.

### ! Important:

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document. Performances under guarantee do not extend the term of guarantee.

### 9. Disposal in accordance with EU guideline 2002/96/EU

 This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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