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9.1 Registrierungsnummer

WEEE-Reg.-Nr. DE 75542610

GUARANTEED TOP QUALITY THROUGHOUT

English

User Information

FINO Stereo Microscopes 86000
Stand
Spring-loaded hinged arm 86001

By purchasing this product you have made an excellent choice.

FINO quality products from DT&SHOP convince through consistently high quality for economical prices.

1. FIELDS OF APPLICATION

FINO Stereo Microscopes are used for the surface examination in different fields of dental technology.

2. ADVANTAGES

- ◆ High quality standard
- ◆ Technical precision
- ◆ Adjustable eye distance
- ◆ Fatigue-free working

3. PHYSICAL DATA

FINO Stereo Microscope, spring-loaded hinged arm

Infinitely adjustable, stable hinged arm made of metal, 850 mm, with three lock screws in the spring-loaded hinges, broad field ocular pair WF 10x, rotational revolver 1x and 2x for magnification 10x/20x, top-down fluorescent ring tube 230 V/9 W/170 mA electronic intermediate unit with 2 m supply cable.

FINO Stereo Microscope, stand

Metal stand: stand length 180 mm, flat base 200 x 150 mm, broad field ocular pair WF 10x, rotational revolver 1x and 2x for magnification 10x/20x, top-down fluorescent ring tube 230 V/9 W/170 mA electronic intermediate unit with 2 m supply cable.

4. PROCESSING / INSTRUCTIONS

4.1 Commencement of operation

Please check the unit for transportation damages immediately after unpacking. All transportation damages have to be claimed promptly. The tabletop clamp respectively the tabletop base is attached to the table and the hinged arm is put into the socket. Fixation is done with the lateral screw.

The included spring washer is interposed before the microscope head is mounted. Thus fixation of the microscope head is ensured. The top-down illumination is attached to the microscope in a way that the working area is best possibly illuminated. After checking that the voltage on the type label of the unit corresponds with the available mains supply the plug of the unit is connected to the power socket.

4.2 Operation

The spring-loaded hinged arm is adjusted to the desired position to ensure relaxed working. Now the lens magnification 1x or 2x can be selected. The eye distance is individually adjustable. If required the top-down illumination can be switched on and can be adjusted to the object.

4.3 Maintenance

FINO Stereo Microscopes are maintenance-free. Repairs of the unit should only be done by DT&SHOP or an authorised workshop. The user has to stand good for damages caused by unauthorized/ arbitrary repairs. The unrestricted conditions of delivery and guarantee of DT&SHOP apply for this unit.

5. STORAGE

The FINO Stereo Microscopes should be installed in a potential dust-free area and should be covered after use.

6. SAFETY PRECAUTIONS

None

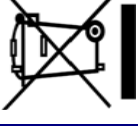
7. AVAILABILITY

FINO Stereo Microscope, spring-loaded hinged arm each 86000
 FINO Stereo Microscope, stand each 86001

8. GUARANTEE

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

9. DISPOSAL IN ACCORDANCE WITH EC GUIDELINE 2002/96/EG

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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