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90109

Deutsch

### Gebrauchsinformation

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### User Information

#### 1. Fields of application

FINOTIMER is an exact timing device for measuring time intervals between 1 second and 99 minutes.

#### 2. Advantages

- Easy handling
- Large LCD display
- Universally to attach

#### 3. Safety precautions

Flat batteries must be removed from the device. They are regarded as hazardous waste and must not be disposed with household waste but at specifically designated collection points. Used batteries must be removed from the device. They are regarded as hazardous waste and must not be disposed with domestic waste but at specifically designated collection points.

#### 4. Processing instructions

The FINOTIMER is quick and easy to operate and is thus ideal for surgery and laboratory. The timer has a large, well readable LCD. A magnet on the back enables an easy attachment to metal surfaces. The integrated stand also allows a safe positioning of the FINOTIMER at the working place.

#### 4.1 Commencement of operation

Please check the unit for transport damages immediately after unpacking.

Transport damages have to be claimed promptly. Open the battery compartment and insert the battery.

#### 4.2 Operation

##### 4.2.1 Count down

The button "MIN" is for setting the minutes, the button "SEC" for setting the desired seconds. To activate the timer function, push the "START/STOP" button and the adjusted time starts to run.

##### 4.2.2 Count up

Reset the display to 0 by simultaneously pressing the buttons "MIN" and "SEC". The count up starts after pressing the button "START/STOP". For a stop, press "START/STOP". Continue the count up by pressing "START/STOP" again. To end the function, press "START/STOP".

#### 5. Storage

The FINOTIMER should be stored in a dry place above freezing level and not above 50 °C.

#### 6. Technical data

Measuring range: 1 sec.-99 min.  
Voltage: 1.5 V battery  
Weight: 85.5 g

#### 7. Delivery forms

FINOTIMER exact timer  
including 1,5 V battery  
each 90109

#### 8. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures.

Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

#### 9. Disposal in accordance with EU guideline 2002/96/EU

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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