Modellhartgips

REF

FH1001/FH1002

Deutsch

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English

Operating Instructions

Please read carefully before use
Keep for referral purposes

1. Intended purpose

Special gypsum for use in dental technology.

2. Indication

Type III. Production of anatomic casts, counter bite and repair models. Investing in prosthetics.

2.1 Contraindication

Do not use if incompatibilities or allergies to constituents have been confirmed.

3. Fields of application

For processing in the dental laboratory according to the intended use.

4. Safety precautions

Avoid the formation of dust during processing.

Wear dust mask!

P305+P351 IF IN EYES: Rinse cautiously with water for several minutes.

P337+P313 If eye irritation persists: Get medical advice/attention.

P301+P314 IF SWALLOWED: Get medical advice/attention if you feel unwell.

5. Processing instructions

Observe mixing ratio (see 7.). Intersperse plaster powder slowly in the water. Observe dwelling time of approx. 20 s. Subsequently mix the material manually with a spatula. Now mix under vacuum for 30 s. mechanically. Pour impression under slight vibration. Do not vibrate the poured impression any longer to prevent interference with the crystallisation process.

6. Storage

Store in tightly sealed container protected from humidity. Can be stored for a minimum of one year. Fill into smaller containers only in dry rooms.

7. Physical data

Mixing ratio: 100 g powder : 30 ml water; processing time 5-6 min.; setting time 10-12 min.; setting expansion after 2 h 0,17 %, compressive strength after 1 h 23 N/mm², after 24 h 50 N/mm².

8. Delivery forms

FINOHIT model hard plaster

yellow

25 kg

FH1001

blue

25 kg

FH1002

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

10.1 EWC waste code product

170802 CONSTRUCTION AND DEMOLITION WASTES (INCLUDING EXCAVATED SOIL FROM CONTAMINATED SITES); gypsum-based construction material; gypsum-based construction materials other than those mentioned in 17 08 01.

11. Symbols explanation

11.1 Manufacturing

 Item numbers

 Lot code

 Shelf life

 non-sterile

 Unit in the packaging

 Manufacturer

 Date of manufacture

11.2 Application

 Please read carefully before use

Keep for referral purposes

 Only to be used by qualified personnel!

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