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## ENG

## Operating Instructions

Please read carefully before use  
Keep for referral purposes

**1. Intended purpose**

Special gypsum for use in dental technology.

**2. Indication**

Type IV. Production of tooth arches and master casts in the crown and bridge technique and model casting technique.

**2.1 Contraindication**

Do not use if incompatibilities or allergies to constituents have been confirmed.

**3. Fields of application**

For processing in the dental laboratory according to the intended use.

**4. Safety precautions**

Avoid the formation of dust during processing.

Wear dust mask!

P305+P351 IF IN EYES: Rinse cautiously with water for several minutes.

P337+P313 If eye irritation persists: Get medical advice/attention.

P301+P314 IF SWALLOWED: Get medical advice/attention if you feel unwell.

**5. Processing instructions**

Observe mixing ratio (see 7.). Intersperse plaster powder slowly in the water. Observe dwelling time of approx. 20 s. Subsequently mix the material manually with a spatula. Now mix under vacuum for 30 s. mechanically. Pour impression under slight vibration. Do not vibrate the poured impression any longer to prevent interference with the crystallisation process.

**6. Storage**

Store in tightly sealed container protected from humidity. Can be stored for a minimum of one year. Fill into smaller containers only in dry rooms.

**7. Physical data**

Mixing ratio: 100 g powder : 20 ml water; processing time 6-7 min.; setting time 12-14 min.; setting expansion after 2 h 0,09 %; compressive strength after 1 h 25 N/mm<sup>2</sup>, after 24 h 60 N/mm<sup>2</sup>.

**8. Delivery forms**

FINOHIT super hard plaster  
golden brown

25 kg

FH1005

**9. Guarantee**

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

**10. Disposal**

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

**10.1 EWC waste code product**

170802 CONSTRUCTION AND DEMOLITION WASTES (INCLUDING EXCAVATED SOIL FROM CONTAMINATED SITES); gypsum-based construction material; gypsum-based construction materials other than those mentioned in 17 08 01.

## 11. Symbols explanation

### 11.1 Manufacturing

	Item numbers
	Lot code
	Shelf life
	non-sterile
	Unit in the packaging
	Manufacturer
	Date of manufacture

### 11.2 Application

	Please read carefully before use Keep for referral purposes
	Only to be used by qualified personnel!
	Do not reuse
	Biohazard

### 11.3 Storage

	Temperature limit
	Keep away from direct sunlight
	Protect against moisture
	Do not use damaged packaging
	top - store upright
	Caution fragile

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