Deutsch

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FINOHIT Silikonknetmasse  
FINOHIT Härterpaste

FH1132/FH1134  
FH1130/FH1131

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**FINOHIT Silicone Putty**  
**FINOHIT Hardener**

**FH1132/FH1134**  
**FH1130/FH1131**

**Please read carefully before use**  
**Keep for referral purposes**

### 1. Fields of application

Condensation curing. For producing moulds, repair models, counter models in relining units, bite and shape registrations for modelling.

Only for extraoral use!

### 2. Safety precautions

Silicone putty:

No particular hazards worth mentioning.

Hardener:

EUH208 Contains dibenzoyl peroxide. May produce an allergic reaction.

### 3. Processing instructions

#### 3.1 Dosing

Dose a circular measuring spoon of silicone modelling material and wipe off excess. Spread dosed amount on hand and dose a strand of curing paste per circular diameter. Mixing ratio silicone modelling material : curing paste 1 measuring spoon : 4 cm.

#### 3.2 Mixing

Fold the silicone modelling material and knead both components well for approx. 1.5 min until a homogeneous mixture (colour) has been achieved. Process the silicone modelling material.

#### 3.3 Curing

Curing time approx. 5 min at room temperature. Curing can be accelerated or slowed down through over or underdosing of the curing agent.

The curing time is accelerated by longer kneading.

### 4. Storage

P404 Store in a closed container.

P411 Store at temperatures not exceeding 23 °C/50 °F.

P410 Protect from sunlight.

P273 Avoid release to the environment.

## 5. Physical data

Mixing time 1.5 min; setting time 5 min; hardness after 60 min. 95 Shore A; colour silicone modelling material (base): blue, colour curing paste (catalyst): red.

### 5.1 Composition

Silicone putty mainly consisting of organic derivatives and polydimethylsiloxane.

## 6. Delivery forms

FINOHIT silicone putty	1600 g	FH1132
	10000 g	FH1134
FINOHIT hardener	40 ml	FH1130
	5 x 40 ml	FH1131

## 7. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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