Deutsch

## Gebrauchsinformation

**FINOHIT** Lichthärtegerät

**FH2305**

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**FINOHIT** Light-curing Unit

**FH2305**

By purchasing this product you have made an excellent choice. FINOHIT quality products from **FINO** convince through consistently high quality at economical prices.

### 1. FIELD OF APPLICATION

Light-curing unit designed for the curing of light-curing materials in the dental field in particular for light-curing tray materials, sealing varnishes and block-out materials.

### 2. TECHNICAL DATA

Voltage:	230 V; 50 Hz
Consumption:	18 Watt
External dimensions:	26,5 x 22,0 x 10,0 cm
Polymerisation chamber:	150 x 150 x 50 mm
UVA range:	350 – 400 nm
Type of tubes:	2 x Philips PL-S 9W/10/2P
Durability of tubes:	approx. 2000 h
Hours of operation respectively trippings:	approx. 20.000

### 3. DESCRIPTION

Install the included tubes in the unit. Therefore remove the drawer of the polymerisation chamber. The drawer can be completely removed by pushing the unlocking button on left inner side of the polymerisation chamber. Place the tubes in the sockets. After pushing slightly you will feel the tubes catch. To replace the drawer put it precisely fitting on the rails and push in evenly.

Check the voltage given on the type label of the unit and make sure it corresponds with your mains supply.

Is this the case you can connect the unit plug with a grounded socket.

Switch the unit on using the main switch. It is located on the rear side of the unit.

Open the drawer of the polymerisation chamber and place the object to be cured there.

Close the drawer and select the curing time by pushing the timer on the upper side of the unit. Different selections are possible:

1 x short push 5 minutes

1 x >push for 2 seconds continuous operation.

The process can at any time be interrupted by again pushing the timer.

The drawer can be opened and closed at any time without affecting the cycle of the selected curing time.

### 3.1 Information

Do never operate the unit with wet hands, you are risking an electrical stroke Unplug the unit from the socket before you change the tubes.

## 4. AVAILABILITY

<b>FINOHIT</b> Light-curing Unit	each	FH2305
FINOLUX Tube	each	14955

## 5. GUARANTEE

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures.

Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products

## 6. TROUBLESHOOTING

After some hours of operation the unit switches off and cannot be restarted:

- Let the unit cool down for some minutes.

The tubes do not work:

- Change the tubes.

If the above-mentioned measures are not successful or if other problems occur that are not covered here please contact the technical customer service.

## 7. DISPOSAL IN ACCORDANCE WITH EC GUIDELINE 2002/96/EG

This equipment may not be disposed in domestic waste!  
Please contact the manufacturer or your dealer!

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