Deutsch

Gebrauchsinformation

FINOHIT Ultraschallgerät

FH6750

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FINOHIT Ultrasonic Unit

FH6750

By purchasing this product you have made an excellent choice. FINOHIT quality products from **FINO** convince through consistently high quality at economical prices.

1. FIELDS OF APPLICATION

Small but stable ultrasonic unit for the cleaning of dentures, spectacles and other small objects. It reliably removes residues of polishing pastes and other dirt from the objects you want to clean.

2. TECHNICAL DATA

Voltage:	220–240 V
Consumption:	50 W
Cleaning frequency:	42 KHz
External dimensions:	20,3 x 12,7 x 14 cm
Tank size:	15,5 x 9,5 x 3,5 cm
Tank volume:	400 ml

3. PROCESSING / INSTRUCTIONS

3.1 Getting started

Please check the unit for transportation damages immediately after unpacking. If there should be any transportation damages the unit must not be plugged into your mains supply. All transportation damages must be claimed promptly. Check the voltage given on the type label of the unit and make sure it corresponds with your mains supply. In this case you can connect the unit plug with a grounded socket. Choose an even and dry place of installation.

3.2 Operation

Fill the unit with approx. 400 ml water and maybe add a bit of cleaning agent. Place the objects you intend to clean in such a way in the basket that no air bubbles are trapped under the objects. The objects must not be in contact with each other. Close the lid of the unit and switch it on using the ON - switch. The cleaning process can be interrupted at any time with the OFF – switch. The unit switches off automatically after approx. 3 minutes. Repeat the cleaning process if necessary.

The included resin bracket is designed for the cleaning of metal watch wristlets. Place the watch with opened wristlet around the bracket. Make sure that the watch casing does not immerse into the cleaning liquid during cleaning. This could damage the watch. The ultrasonic unit should not be operated for more than 30 minutes continuously. If the cleaning solution is contaminated please dump according to directions of the manufacturer.

3.3 Safety precautions

Do not grasp into the liquid during operation!

Never operate the unit without liquid!

Do never touch the unit with wet or moist hands!

Do not use flammable liquids like benzene as solvent!

Do not place objects on the floor of the tank, always use the immersible basket!

Unplug the unit when it is not in use!

If you clean watch wristlets make sure that the watch casing does not come in contact with the liquid

4. AVAILABILITY

FINOHIT Ultrasonic Unit

each

FH6750

Immersible basket

Watch bracket

5. GUARANTEE

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures.

Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

6. DISPOSAL IN ACCORDANCE WITH EC GUIDELINE 2002/96/EG

This equipment may not be disposed in domestic waste!
Please contact the manufacturer or your dealer!

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